Digital Auscultation Device Nexstetho SDNX-01G Operation Manual

Contents

1. Before use

- 1-1 Check the package contents
- 1-2 For your safety
- 1-3 Terms of Use
- 1-4 A note on wireless communications

2. Names of parts

- 2-1 Power switch
- 2-2 Power indicator
- 2-3 Volume dial
- 2-4 Micro-USB port
- 2-5 ϕ 3.5mm mini jack
- 2-6 Wireless button
- 2-7 Mute button
- 2-8 I button, II button

3. Using the Nexstetho

- 3-1 How to attach to the neck (metal part)
- 3-2 Battery charging
- 3.3 Analog wired connection
- 3.4 USB audio connection

4. Regulations and Safety Precautions

- 4-1 FCC
- 4-2 NCC

1. Before use

1-1 Check the package contents

Package includes the following	
SDNX-01G (main unit)	1 pc.
USB cable	1
adjuster tube	1
O-ring (large)	2 pcs.
O-ring (small)	2 pcs.
Hexagonal wrench	1 pc.
Operation Manual (this manual)	1 copy

1-2 For your safety


- Do not attempt to disassemble, repair or modify this product by yourself. Do not attempt to disassemble, repair or modify this product for safety reasons. Do not disassemble, repair or modify this product.
 - Do not use this product in a wet or humid environment. This product is not waterproof and should not be used in wet or humid places.
 - This product is equipped with a lithium-ion battery. Do not throw it into a fire or give it a strong shock. It may cause an explosion, electric shock, or malfunction.
 - Do not use the product with the volume set to the highest setting, which may cause earache. Use at a moderate volume.
 - Please stop using it immediately if you feel abnormalities and consult with your dealer.
- #### 1-3 Terms of Use
- We are not responsible for any damage caused by the use of this product and software.
 - Do not use this product in a humid environment.
 - Do not expose this product to strong shocks.

- Do not use this product in a humid environment. Do not use benzene, thinner or alcohol.
- In rare cases, the mode setting may be canceled when exposed to static electricity, but this does not affect the function of the main unit. Please set the mode again.


1-4 A note on wireless communications

- This product is compatible with the 2.4 GHz band short-range wireless communication standard for digital devices.
- This product is compatible with the 2.4 GHz band short-range wireless communication standard for digital devices. If this product is used in the vicinity of wireless LAN or other wireless devices, in the vicinity of microwave ovens or other devices that emit radio waves, in places with many obstacles, or in other environments where the radio wave conditions are poor, there is a possibility that the connection may be frequently interrupted or an error may occur.
- Please be aware that there is no guarantee of compatibility with all devices in terms of communication standards.


2. Names of parts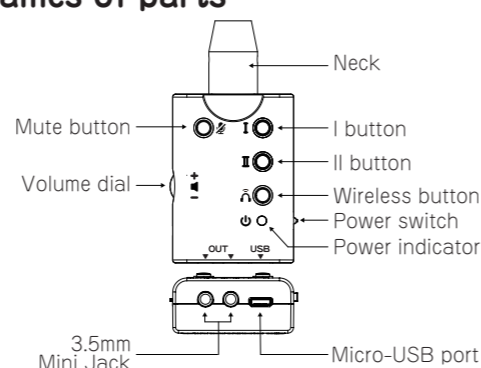



If the diameter of the connection is too small, use the supplied adjuster tube.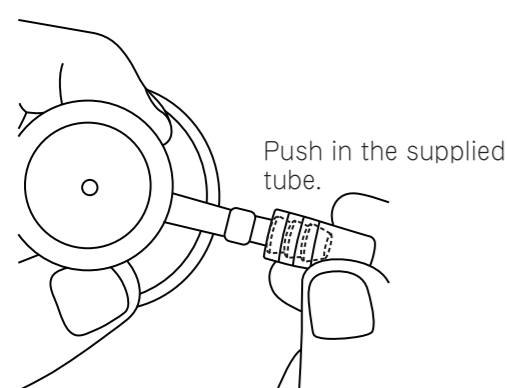


If necessary, use an O-ring small on top of the adjuster tube to adjust the diameter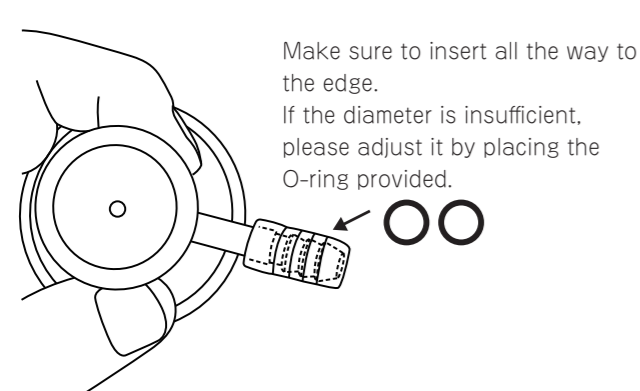


Fasten the screws already installed in the main unit with the wrench provided on both sides.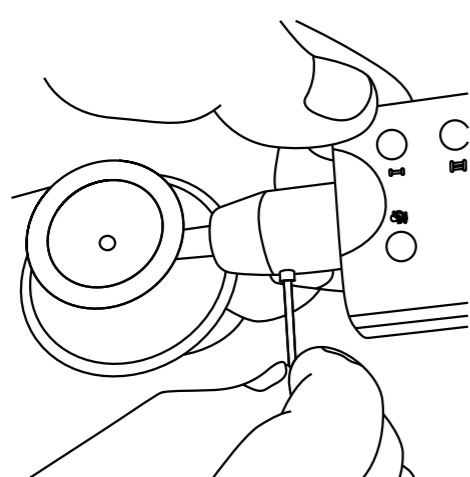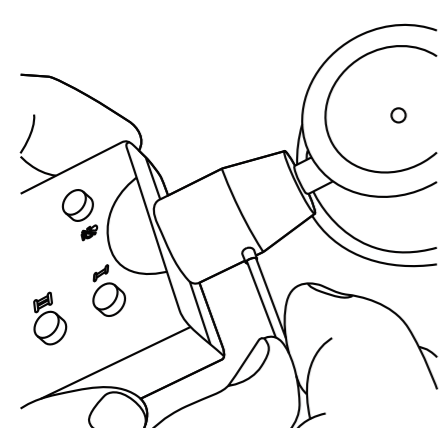

In the case of the double-head type, be careful not to make a mistake in the direction of the hole opening.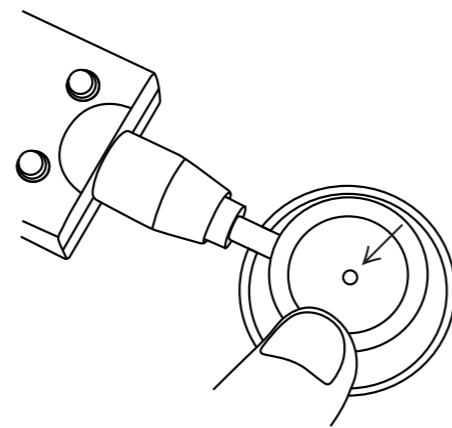

Slowly insert it into the neck of the Nexstetho.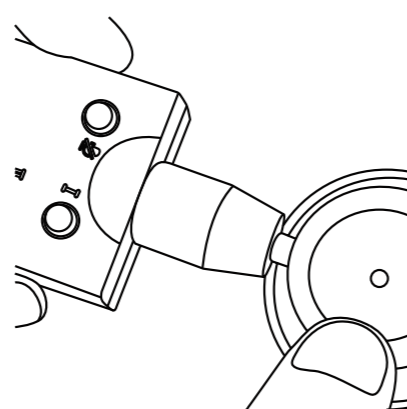

3-2 Battery charging

- 1 Charge the battery before using the product.
 - 2 Please turn the power switch to the OFF position.
 - 3 Connect to the MicroUSB port using the supplied Micro-USB cable and charge the battery using a commercially available charging adapter with 1A output.
- Charging takes 2 hours to fully charge. Disconnect the cable from the Micro-USB port and use the product. When fully charged, the product can be used for approximately 4 hours of continuous operation. In addition, the continuous standby time is approximately 250 hours. This product uses a lithium-ion battery.

3-3 Analog wired connection

Connect the 3.5 mm 3-pole audio mini-jack to the audio jack. Two jacks can be used at the same time.

[NOTE] Any device with a microphone or controller with a 4 or 5-pin connector cannot be used.

3-4 USB audio connection

- Smart phone
Use a commercially available OTA adapter to connect to your smartphone with the attached Micro-USB cable.
 - PC
Connect the attached Micro-USB cable to your computer.
- [NOTE] We do not guarantee everything.

2-1 Power switch

Turn on the power: slide the switch up (power indicator lights up green)

Turn off the power: Slide the switch down (power indicator is off)
[NOTE] The middle section of the switch is for maintenance purposes only and not for normal use.

2-2 Power indicator

When the power is turned on, it turns on green. When the remaining battery charge is less than 10%, it blinks red, please start charging. The power indicator turns on red while charging, and it turns off when charging is completed.

2-3 Volume dial

Turn up the volume: turn up from the bottom to the top
Turn down the volume: turn from top to bottom.

2-4 Micro-USB port


The Micro-USB terminal is used for charging, recording, maintenance, etc. To charge the battery, connect the supplied cable to the Micro-USB port with the power off, you can use a USB style AC adapter or mobile battery (commercially available) with a minimum of 1A output to charge the battery. The power indicator lights up red while charging and turns off when charging is complete.

2-5 ϕ 3.5mm mini jack


You can connect a 3.5mm 3-pole mini-jack. Two channels can be connected simultaneously.
[NOTE] A mini-jack with a volume or microphone on it cannot be used.

2-6 Wireless button

Pairing Mode (first time only)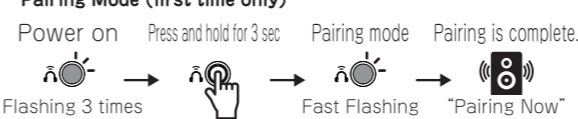


Searching mode (after the second time)

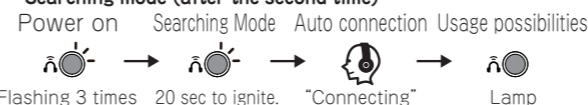
- **The unit will turn on :** the wireless LED will blink three times and turn off. This means that the wireless function has been successfully activated. Wireless is inactive at this point.
- **Pairing :** With the wireless inactive, press and hold the wireless button for 3 seconds. Pairing will begin. Blinking pattern is 0.2 seconds. The LED will be constantly lit blue when connected to a wireless device. Turn off the unit when you are finished pairing.

1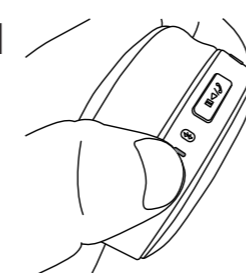

Put the headphones into pairing mode.

2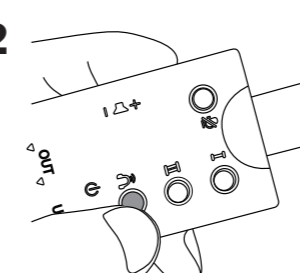

Put the Nexstetho in pairing mode. Press and hold the wireless button for about 3 seconds.

2-7 Mute button

Microphone is ON : LED is off

Microphone is OFF : With one click, the red LED lights up and the microphone is turned off.

[NOTE] Please note that high volume may cause howling.

2-8 I button, II button

By using the I and II buttons, you can use four different sound settings. When the power is turned on, the LED of button I and II is off, and with one click, the LED lights up in green and the sound setting for each mode is turned on.

Normal	Beat	Flow	Jet
I	I	I	I
II	II	II	II


- Normal:** Unadjusted status.
- Beat :** Make low frequencies easier to hear (heartbeat, vibration, etc.)
- Flow:** Easy to hear mid and high frequencies (water flow, air flow, etc.)
- Jet:** High frequencies are easier to hear (e.g., jetting, water jet)

- **Searching :** Press the wireless button once while the wireless is inactive; search will begin for 20 seconds. The blinking pattern is 1 second apart. When connected to a wireless device, the LED will be lit constantly in blue. If it does not connect and you need to search again, press the wireless button again. It works the same way even if the device does not have any wireless device memory.
- **Memory Erase :** Press the wireless button while the wireless is inactive for 8 seconds. Turn the unit off once to use it again.
- **storage devices :** Up to two pairing memories can be stored (but only one can be connected). A new pairing will erase the old one.

3. Using the Nexstetho

3-1 How to attach to the neck (metal part)

If the connection is relatively uneven and the smallest diameter (neck) is about 6-7mm, use the small O-ring. ex) Littmann Cardiology III, IV

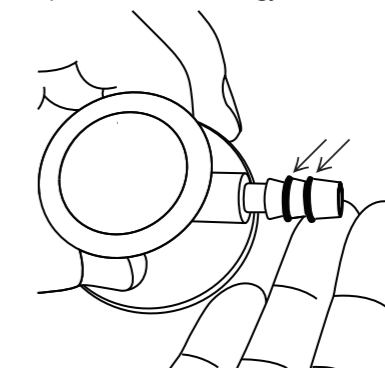4. Regulations and Safety Precautions

4-1 FCC

FCC (U. S. Federal Communications Commission)

This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions:

1. This device may NOT cause harmful interference.
2. This device must accept any interference received, including interference that may cause undesired operation.

FEDERAL COMMUNICATIONS COMMISSION INTERFERENCE STATEMENT

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. The limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one of the following measures:

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- Reorient or relocate the receiving antenna.
- Increase the separation between equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

CAUTION:

Any changes or modifications not expressly approved by the grantee of this device could void the user's authority to operate the equipment

The changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

RF exposure warning

This equipment must be installed and operated in accordance with provided instructions and the antenna(s) used for this transmitter must be installed to provide a separation distance of at least 20 cm from all persons and must not be co-located or operating in conjunction with any other antenna or transmitter. End-users and installers must be provide with antenna installation instructions and transmitter operating conditions for satisfying RF exposure compliance.

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